



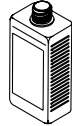

assistina

Weekly checklist


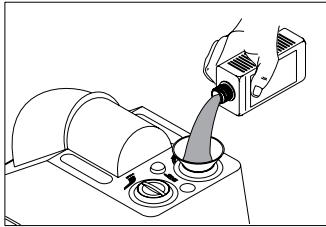


**Check liquid level!
Refill if necessary!**

Service oil:
500 ml – 02675910


Cleaning solution:
1000 ml – 02680230

Set 02680500:
1000 ml Cleaning solution and 500 ml Service Oil

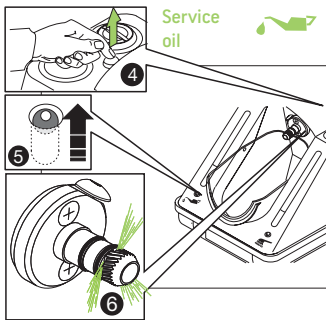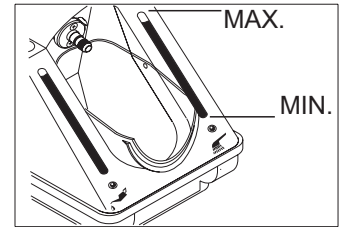

Service
oil


Check function!

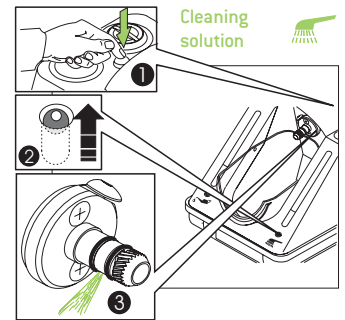

Cleaning
solution

Check o-rings!


**Look for right position
and / or
damage!**

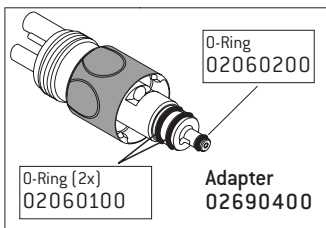

O-Ring
02060200

O-Ring (2x)
02060100

Adapter
02690400

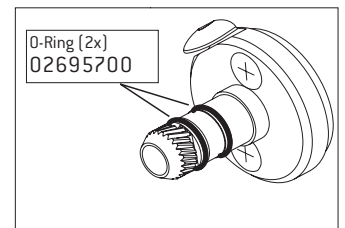


O-Ring (2x)
02695700



General hints



*When maintaining after every patient – Press program button once!
When maintaining twice a day – Press program button twice!*

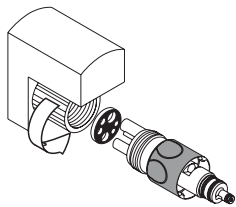

Check connection is tight!

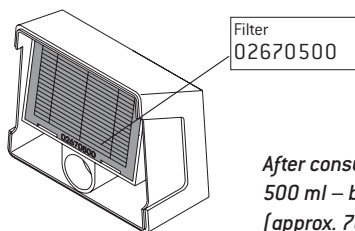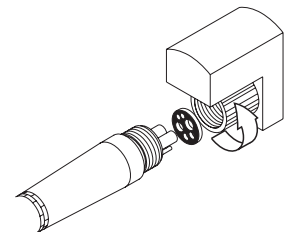

Filter
02670500

*After consumption of one
500 ml – bottle of Service oil
(approx. 7000 cycles)*

Changing of filters!



Filter
02675200

Once a year